


## ***How do I do to add a new version of the Program to Gewa Progress?***

Add a new version in Progress by the following procedure:

1. Make a back-up copy in order to save the settings, see the section "Making a back-up" in the manual.
2. Remove the pocket-PC from Progress.  
Reset the pocket-PC. The program now restarts and enters Windows.
3. Install ActiveSync from the CD that was delivered with Progress. See the instructions from HP: "Getting started".
4. Connect the accompanying docking station to a USB port or the serial port on the PC.
5. Place the pocket-PC into the docking station. An installation guide will appear called "New partnership". Select "Guest partnership" and then click on "Next".
6. Click on the installation file that you have received via e-mail or downloaded from Gewa's website.
7. Follow the instructions in the installation program. Answer "Yes" to the questions about "reinstall/upgrade" and installation on "default application install directory". Now information about the installation is transferred to the pocket-PC.
8. After the transfer a dialogue box is shown on the pocket-PC. Answer OK to the question "Reinstall"? Then answer "Yes to All" to the question "replace existing file?". The installation files are now transferred to the memory card on the pocket-PC.
9. Remove the pocket-PC from the docking station and put it back in the Progress Module.
10. Remove the memory card and put it back again. The update now starts. A dialogue box is shown on the pocket-PC. Answer "OK" to the question "Reinstall"? Then answer "Yes to All" to the question "replace existing file"? Progress automatically restarts after the installation.
11. Confirm the update by looking at the version number on the top of the page you reach after the restart or go to "Settings/System/Version" (Progress X.X.X.).

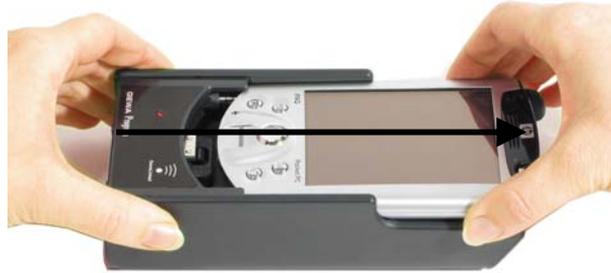

# **Gewa AB**

BOX 92, SE-191 22 SOLLENTUNA, SWEDEN

TEL: +46 (0)8-594 694 00 • TEXTTEL: +46 (0)8-594 694 18 • FAX: +46 (0)8-594 694 19

E-MAIL: [info@gewa.se](mailto:info@gewa.se) • WEB: [www.gewa.se](http://www.gewa.se)